

# Disposable hemostatic clips

## Instruction for Use

**【Product name】** Disposable hemostatic clips

**【Product description】**

The main components of Disposable Hemostatic Clips are Clip (07Cr17Ni7Al), clip sleeve (SUS316F), rotating sleeve (SUS316F). The conveying part is mainly composed of spring tube (06Cr19Ni10), plastic coating (PE), tapered rod head (ABS), sliding rod (ABS), Slip ring (ABS), thumb ring (ABS),

The Disposable Hemostatic Clips sterilized using Ethylene Oxide. Shelf life is 2 years. Only single use.

**【Specification and type】**

ATE-ZXJ-A-2.3X1250X90(135)X9(11, 13, 16)

ATE-ZXJ-A-2.3X1650X90(135)X9(11, 13, 16)

ATE-ZXJ-A-2.3X1950X90(135)X9(11, 13, 16)

ATE-ZXJ-A-2.3X2350X90(135)X9(11, 13, 16)

ATE-ZXJ-B-2.3X1250X90(135)X9(11, 13, 16)

ATE-ZXJ-B-2.3X1650X90(135)X9(11, 13, 16)

ATE-ZXJ-B-2.3X1950X90(135)X9(11, 13, 16)

ATE-ZXJ-B-2.3X2350X90(135)X9(11, 13, 16)

**【Usage】** Used with endoscope, used for soft tissue mechanical sandwich with endoscope hemostasis.

**【Method of use】**

According to the actual need to choose the appropriate product model, open the single packing, will be a one-time use of hemostatic clip by clip head first into endoscopic pliers, within the tao by

endoscope and aim the chuck of soft tissue into the lesion. Push open slip ring clamp head near the clip sco, pull the folded slip ring clip. To pull back slip ring clamp holder and the circlip interlock, the hemostatic clamp two clip occlusion soft tissue, hemostatic clamp two clamp body separation. Twitch handle pull the body and bourdon tube under endoscope pliers.

**【Matters needing attention】**

1. The operator of the use of this product must be a medical personnel, and must have the corresponding qualification;
2. This product by the EO sterilization, sterile products, valid for two years after the sterilization of qualified, recognizing when using single packing pincer-like device failure, overdue it is prohibited to use;
3. The single packing damage, spare parts, packaging banning the use of foreign body in it, and destroy;
4. This product is only for one-time use, destroyed after use.



**【contraindications】**











1. acute perforation of esophagus, stomach and duodenum.
2. serious heart, lung, kidney, brain dysfunction and multiple organ failure.
3. psychosis, and impaired consciousness obviously can't collaborator.
4. and other specific by professional doctor according to the actual condition of the patient.

**【Packaging and storage】**

1. Single packaging uses paper-plastic package; middle packaging uses cartons; external packaging uses corrugated carton.
2. Products after packing should be stored in a well-ventilated room with relative humidity not more than 80% and no corrosive gas.

**【Marking instruction】**


Symbol	Title of Symbol	Symbol	Title of Symbol
	Manufacturer		Authorized representative in the European Community

	CE marking		Date of manufacture
	Catalogue number		Batch code
	Use-by date		Sterilized using ethylene oxide
	Do not re-use		Caution
	Working diameter		Logo



Jiangsu Ate Medical Technology Co.,Ltd.  
 No.8, Lanxiang Road, Wujin Economic Development Zone, 213161 Jiangsu,  
 China

Tel: 0086-519-85288988 Fax: 0086-519-85288060

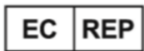

ZOUSTECH S.L.  
 Pso. Castellana, 141 - Planta 19, 28046 - Madrid, Spain  
 Tel: + 34694426446 Fax: + 34917915466



TÜV Rheinland LGA Products GmbH  
 Tillystraße 2, 90431, Nürnberg, Germany

Document number: ATE/IFU-ZXJ-0001, version: A/1