

# Disposable Spray Pipes

## Instruction for Use

**【Product name】** Disposable Spray Pipes

**【Product description】**

Disposable Spray Pipes is composed of nozzle, outside the tubes and handles, etc. Made from nozzle with 00Cr18Ni14Mo3, outer tube using PTFE, handle is made with PC. According to the nozzle type divided into mist and two straight line shape, W represents mist nozzle, Z line shape of nozzle.

The Disposable Spray Pipes sterilized using Ethylene Oxide. Shelf life is 2 years. Only single use.

The Disposable Biopsy Forceps sterilized using Ethylene Oxide. Shelf life is 2 years. Only single use.

**【Specification and type】**

ATE-PSG-10-500(1200, 1600, 1800, 2000, 2300)-W(Z)

ATE-PSG-18-500(1200, 1600, 1800, 2000, 2300)-W(Z)

ATE-PSG-23-500(1200, 1600, 1800, 2000, 2300)-W(Z)

**【Usage】** The device is intended for use in conjunction with endoscope in irrigation and spraying medicine.

**【Method of use】**

Open single packing, the nozzle and the outer tube into endoscopic clamp the tao and into the body's natural cavity lavage or spray liquid operation.

**【Matters needing attention】**

1. The operator of the use of this product must be a medical personnel, and must have the

corresponding qualification;

2. This product by the EO sterilization, sterile products, valid for two years after the sterilization of qualified, recognizing when using single packing pincer-like device failure, overdue it is prohibited to use;
3. The single packing damage, parts fall off, single packing banning the use of foreign body in it;
4. This product is only for one-time use, destroyed after use.









**【Contraindications】**




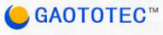
1. Acute perforation of esophagus, stomach and duodenum.
2. Serious heart, lung, kidney, brain dysfunction and multiple organ failure.
3. Psychosis, and impaired consciousness obviously can't collaborator.
4. And other specific by professional doctor according to the actual condition of the patient.

**【Packaging and storage】**

1. Single packaging uses paper-plastic package; middle packaging uses cartons; external packaging uses corrugated carton.
2. Products after packing should be stored in a well-ventilated room with relative humidity not more than 80% and no corrosive gas.

**【Marking instruction】**


Symbol	Title of Symbol	Symbol	Title of Symbol
	Manufacturer		Authorized representative in the European Community
	CE marking		Date of manufacture
	Catalogue number		Batch code
	Use-by date		Sterilized using ethylene oxide

	Do not re-use		Caution
	Working diameter		Logo



Jiangsu Ate Medical Technology Co.,Ltd.  
 No.8, Lanxiang Road, Wujin Economic Development Zone, 213161 Jiangsu,  
 China

Tel: 0086-519-85288988      Fax: 0086-519-85288060

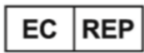

ZOUSTECH S.L.  
 Pso. Castellana, 141 - Planta 19, 28046 - Madrid, Spain  
 Tel: + 34694426446      Fax: + 34917915466



0197

TÜV Rheinland LGA Products GmbH  
 Tillystraße 2, 90431, Nürnberg, Germany

Document number: ATE/IFU-PSG-0001, version: A/1